


SQ.line® KERRISON bone punches

So Clean!


1 | Disassembly



1 Close the front handle.

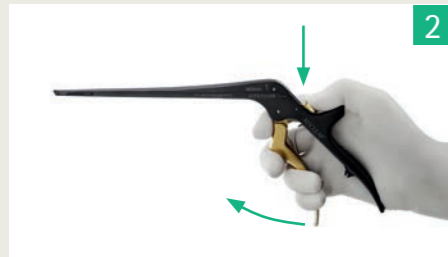

2 Press down the golden latch completely and release the handle.

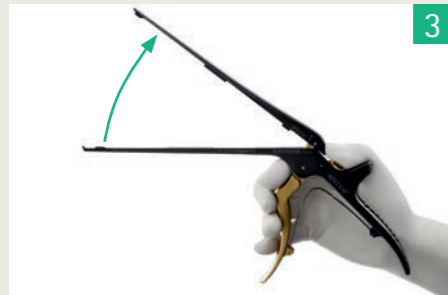

3 Open up the slider.

2 | Cleaning



Place the punches in an open position in the tray for cleaning.



Alternatively place the punches in the SQ.line® KERRISON rack for cleaning (JF125R or JF126R).

3 | Maintenance



The moving parts (hinges, latches and contact area of sliding part) should be lubricated with a sterilizable and moisture-permeable light weight machine oil such as Sterilit® maintenance oil (JG598 or JG600).




4 | Assembly



1 Close the slider.



2 Close and release the front handle.

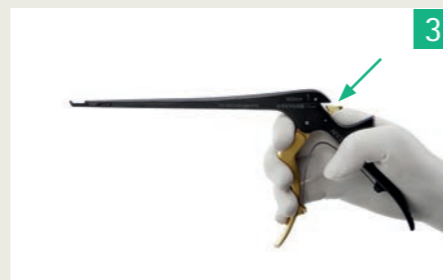

3 The golden latch will close by itself.

5 | Function control



Check that the punches functions correctly. For further information please contact the Aesculap Technical Service.

☎ Phone +49 7461 95-1602

✉ repair.instruments@aesculap.de

6 | Storage



Store the punches in a closed position on the tray or alternatively in the SQ.line® KERRISON rack (JF125R or JF126R).

7 | Sterilization



For further information please see our brochures:

- C72902 SQ.line® KERRISON
- C84802 Proper Maintenance of Instruments
- C63402 Aesculap InOS® – Instrument Organisation System Validated Reprocessing Procedure



Please note the instructions for use (TA018250).

<https://eifu.bbraun.com/eifu/TA018250>